


nuance | Voice Selector
hearing | Converse

Conversation made easy


Getting started

- ① Wear the earphones and press the power button
You'll hear: "Power on"

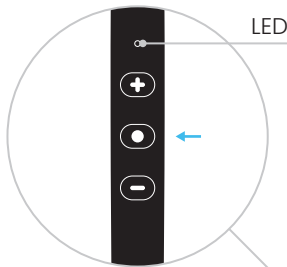


Indications:


- **Blinking red**
Searching for device
- **Blinking blue**
Device found
You'll hear: "Connected"

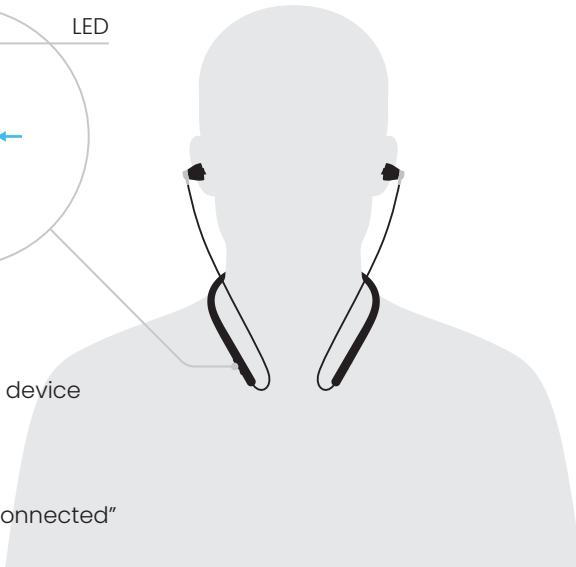

- ② Press the power button

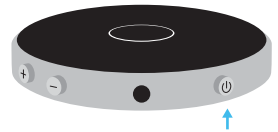


Indications:

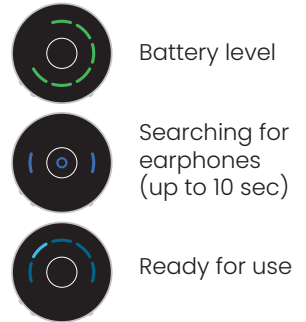

Group conversation


Voice Selector Converse scans the open directions and picks out the dominant speaker automatically

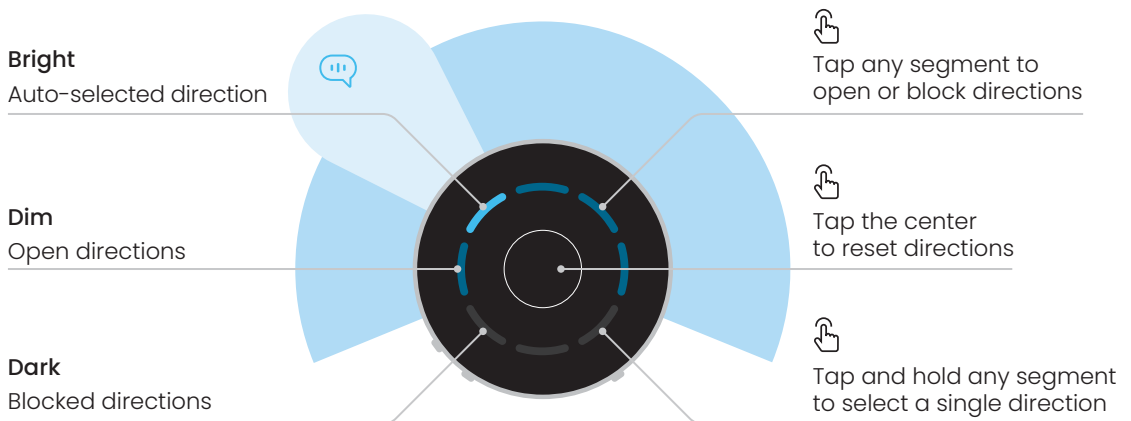

Device orientation: open segments face the group, blocked segments face you


TV Streaming

Enjoy listening to the TV in a comfortable volume level

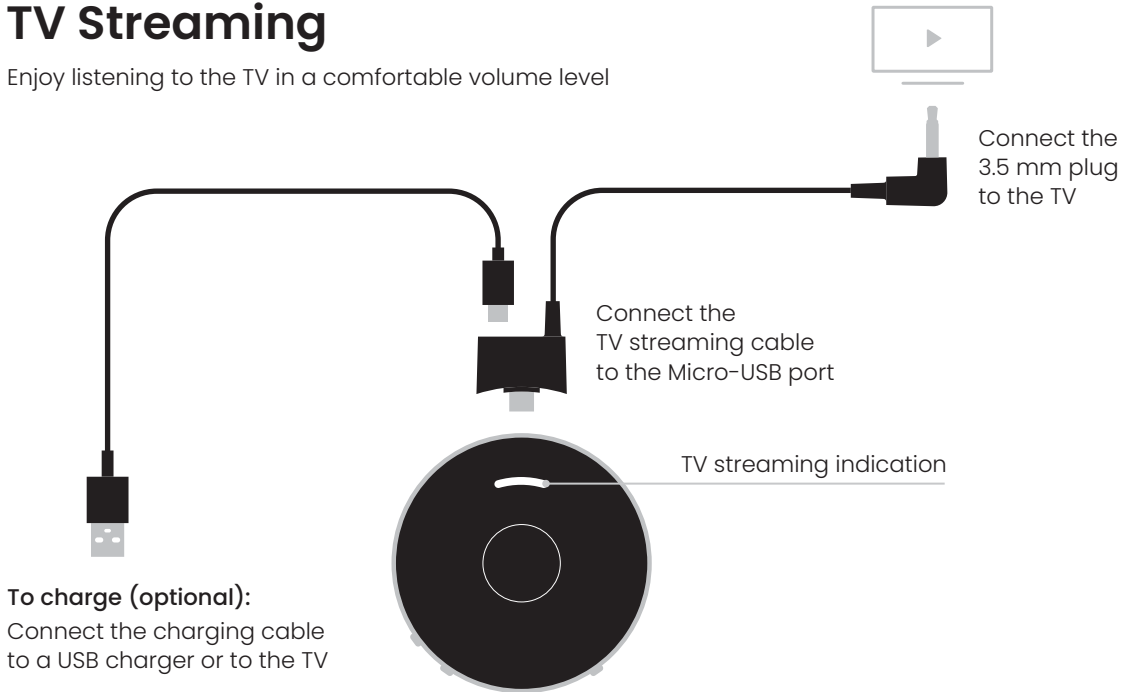


One-on-one conversation

Let your partner wear Voice Selector Converse for optimal clarity

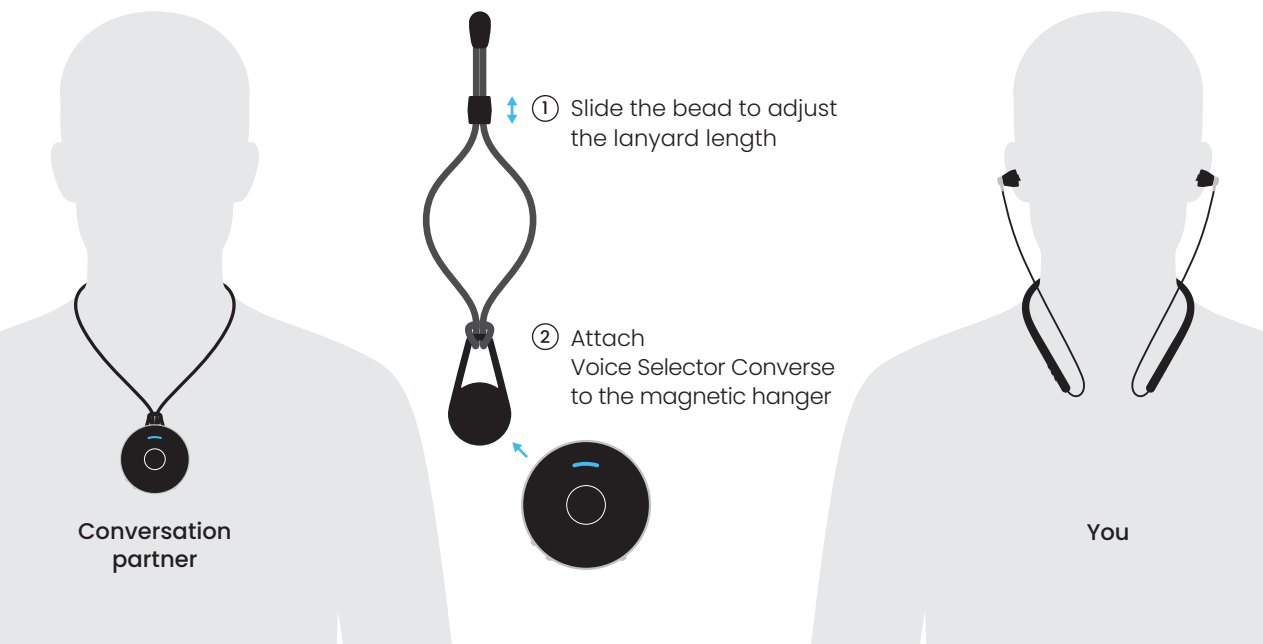


Volume adjustment

For optimal performance, set volume level so voices sound natural (high volume may amplify undesired voices and background noise)

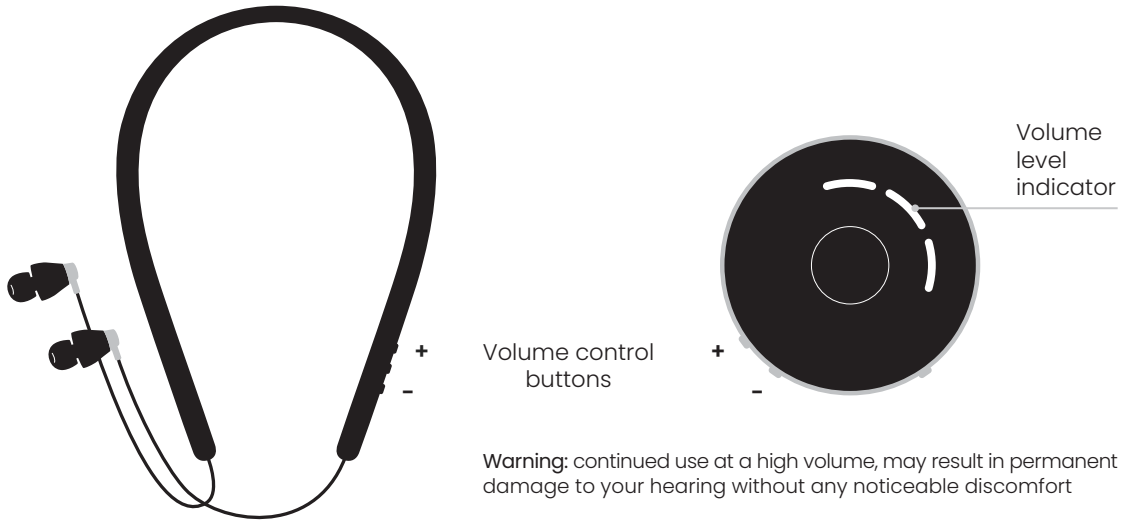

Charging devices

Use the Micro-USB cable to charge your Voice Selector Converse and earphones

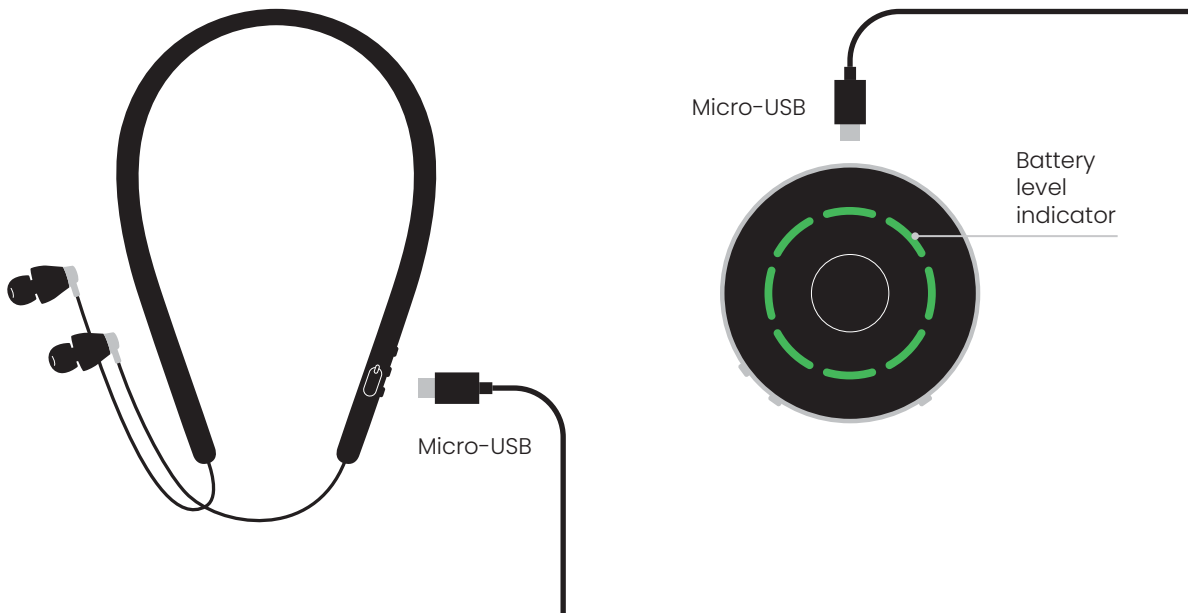


Earphone fitting

Choose the most comfortable fit for your ears

To remove tip, pull firmly


To attach tip, push until it clicks

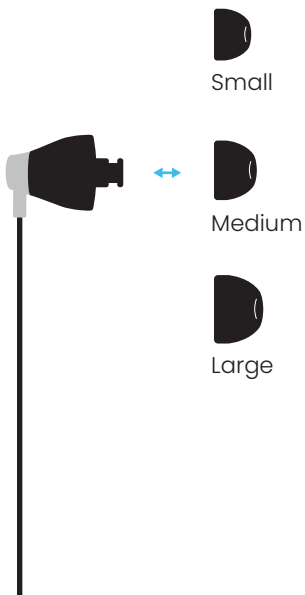


A good seal will reduce outside noise and improve speech understanding

Voice Selector Converse app

Download the app to unlock additional features

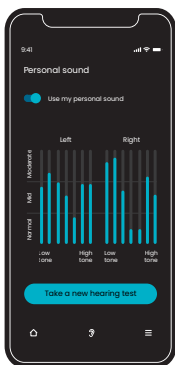

Take a hearing test to create a personal hearing profile

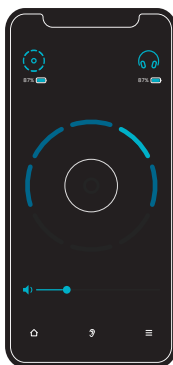

Use the app as a remote control for the device



Scan the QR code



